

INSTRUCTION MANUAL

Catalog No: C10625

Components: Diamond particles, glycerin, binder

Packaging: 3 g syringe

PRODIAMOND DIAMOND POLISHING PASTE

1- Description

PRODIAMOND is a diamond-containing polishing paste suitable for both oral and extraoral applications. It can be used to polish restoration and natural tooth surfaces.

2- Usagearea / Indications

PRODIAMOND is applicable for all clinical and laboratory composite materials, resin-containing hard restoration fillings, metal, ceramic, and porcelain surfaces.

3- How to Use

Attach the application tip to the syringe. Apply a small amount of paste to the designated area using a material such as felt. It does not require a dry area and can be applied on wet surfaces. The paste can be cleaned off with water directly after application.

4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

info@promida.com.tr www.promida.com.tr

Rev. Tr: 01.11.2022

5- Side effects

There are no known side effects.

6- Warning:

Keep the mouth of the syringe closed after use. Do not use if there is a known allergy to any of its ingredients. Use latex gloves and protective glasses when applying the product. If the paste dries out, you can soften it by adding water. If it freezes due to cold, thaw it in a warm environment before use. It is recommended to use below 10,000 rpm to avoid splashing and overheating.

7- Storage conditions:












Store at room temperature between 5°C – 30°C. Protect from moisture, extreme heat, and cold. The product has a shelf life of 2 years from the date of manufacture.

NOTE:

Our products are intended for use in dentistry and are not suitable for non-professional use. Since the application of these products is beyond our control, the user is fully responsible for their use. We recommend using the product in accordance with the information and warnings provided in the instructions.

**** Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

8- Symbols and Meanings:

	Lot Number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on time		Keep away from sunlight
	Warning: Consult the Accompanying Documents.		Refer to the Instructions for Use.
	Storage temperature range		Notified Body
	Store in a dry place.		