

INSTRUCTION MANUAL

Catalog No: A00121

Components: PC acid phenic and other excipients

Packaging: 10 mL

PRORCDPHENIC ACID FOR ROOT CANAL DISINFECTION

1- Usage Area / Indications

PRORCD is used as an antiseptic in root canal treatment. It is indicated for the removal and cleaning of residual tissue and necrotic material in root canal treatment. It is a powerful antiseptic.

2- Usage

The product is impregnated with cotton or similar fibrous material and placed in the treatment area. After waiting for a while, the cotton is removed, and the treatment is continued.

3- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

4- Side effects

If it leaks into living tissue, it may cause irritation and tissue damage. Formaldehyde compounds do not remain within the pulp boundaries but may infiltrate into soft and hard tissues, causing nerve destruction, allergic responses, paresthesia, and localized

necrosis.

5- Warning:

In case of contact with oral tissue, eyes, or skin, immediately clean the area with plenty of water. Use with a rubber dam to prevent it from touching the mucosa. It can only be used by professional dentist.

6- Storage conditions:

Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold. The period of use is 2 years from the date of manufacture.

NOTE:

Our products are produced for use in dentistry. They are not suitable for use other than by professional dentists. Since the use of the products is under the control of the user other than us, the user is completely responsible for the applications. We recommend using them in accordance with the information and warnings given in the instructions.

**** Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.









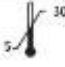


 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

info@promida.com.tr www.promida.com.tr

Rev. Tr: 01.11.2022

7- Symbols and Meanings:

	Lot number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on Time		Keep Away from Sunlight
	Warning: Consult the Accompanying Documents		Refer to the Instructions for Use
	Storage Temperature Range		Notified Body
	Store in a Dry Place		