

## INSTRUCTION MANUAL

**Catalog No:** RE1720

**Components:**

Powder: Zinc Oxide, Resin, and Other Auxiliary Materials

Liquid: Eugenol and Other Auxiliary Materials

**Packaging:** 14 g Powder, 10 mL Liquid.

## PROCANAL G

### 1- Description

PROCANAL G is an endodontic cement used with gutta-percha cones to fill the root canal. It has bactericidal properties and is homogeneously mixed to achieve a creamy consistency that facilitates easy placement of gutta-percha in the canal. The gutta-percha ensures adhesion and sealing within the canal. It does not dissolve after application, does not discolor the tooth, and can be easily removed. It is radiopaque.

### 2- Usage Area / Indications

PROCANAL G is used as an intracanal filling material in endodontic procedures. It is indicated for canal filling techniques using gutta-percha cones.

### 3- Usage

Choose the gutta-percha type. Drop 1-2 drops of the liquid into a mixing glass, add some powder, and mix for 1-2 minutes until a thick, dense, and smooth creamy consistency is achieved. At the ideal density, the spatula will extend about 2 cm when removed from the mass. After completely cementing the gutta-percha, fill the canal according to the appropriate endodontic method.

### 4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

### 5- Side effects

It may cause irritation and has allergenic potential due to the presence of eugenol. It may also cause sedation. Protect the mucosa with a rubber dam during application.

### 6- Warning:

Although it is well tolerated, special care is recommended to avoid overflow onto the periapical tissue. Use a rubber dam to prevent contact with the mucosa.


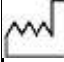







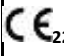

### 7- Storage conditions:

Store at room temperature between 5°C – 30°C. Protect from moisture, extreme heat, and cold. The product has a shelf life of 2 years from the date of manufacture.

**NOTE:**

Our products are intended for use in dentistry and are not suitable for non-professional use. Since the application of these products is beyond our control, the user is fully responsible for their use. We recommend using the product in accordance with the information and warnings provided in the instructions.

**\*\* Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

	Lot Number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on time		Keep Away from Sunlight
	Warning: Consult the Accompanying Documents.		Refer to the Instructions for Use.
	Storage Temperature Range		Notified body
	Store in a dry place.		

**8- Symbols and Meanings:**

 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

[info@promida.com.tr](mailto:info@promida.com.tr) [www.promida.com.tr](http://www.promida.com.tr)

Rev. Tr: 01.11.2022