

KULLANMA TALİMATI

CE

INSTRUCTION MANUAL

Catalog No: A00196

Components: Chlorhexidine Digluconate

Packaging: 250 mL

PROCHEX 2% ChlorhexidineDigluconateIrrigation Solution

1- Description

This solution is used for washing root canals to remove debris.

2- Usage Area / Indications

Accelerates root canal preparation and provides lubrication during mechanical preparation of the root canal.

3- Usage

During mechanical preparation, inject the solution into the root canal using the irrigation tip. Rinse the canal thoroughly with water afterwards

4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5- Side effects

May cause irritation if it comes into contact with living tissues.

PROMIDADIŞMALZEMELERISAN.TIC.LTD.ŞTİ.

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR info@promida.com.tr www.promida.com.tr

Rev. Tr: 06.01.2023

6- Warning:

It may cause irritation if it comes into contact with mucosa. In case of contamination, wash the affected area with plenty of water. It should only be applied by professional dentists. Ensure the bottle caps are always kept tightly closed.

7- Storage conditions:

Store at room temperature between 5°C - 30°C. Protect from moisture, extreme heat, and cold. The product has a shelf life of 2 years from the date of manufacture.

NOTE:

Our products are intended for use in dentistry and are not suitable for non-professional use. Since the application of these products is beyond our control, the user is fully responsible for their use. We recommend using the product in accordance with the information and warnings provided in the instructions.

** **Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.



KULLANMA TALİMAT

CE

8- Symbols and Meanings:

LOT	Lot Number	M	Manufacturing Date
REF	Catalog Number	<u></u>	Manufacturer
23	Use on time	类	Keep away from sunlight
\triangle	Warnin: Consult the Accompanying Documents.	Ţ i	Refer to the Instructions for Use.
5 10	Storage temperature range	C € ₂₂₉₂	Notified Body
*	Store in a dry place.		·

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR info@promida.com.tr www.promida.com.tr

Rev. Tr: 06.01.2023