prēmida®

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Promida Dental Materials, established in 2012, is dedicated to producing dental materials in Turkey that meet world standards. Our product range continues to expand daily, encompassing a wide array of solutions including restorative filling materials, temporary and permanent adhesives, treatment products, and prophylaxis powders and pastes, all essential in various fields of dentistry.

At Promida, we prioritize the health sector's rigorous standards by utilizing globally recognized and proven chemical raw materials in our production processes. Our products undergo meticulous testing by both specialized third-party companies and our in-house laboratory, ensuring they meet the highest levels of safety and efficacy. We value the clinical experiences and feedback from dentists worldwide, which continually inform and enhance our scientific research and product development.

Our commitment to innovation drives our ongoing R&D efforts, allowing us to introduce new products each year. We pride ourselves on our ability to swiftly translate ideas into groundbreaking solutions. This dedication fuels our passion, as we approach each day with the same enthusiasm and commitment to excellence that marked our beginnings.

Our core mission is to consistently deliver quality products that are easily accessible. We adhere to stringent standards throughout our production process.

As we strive to become a global brand, we leverage the expertise of our skilled team, our unwavering commitment to quality, our automation-based production line, and our relentless pursuit of improvement. We are driven by the vision of making Promida a name synonymous with excellence in dental materials worldwide.

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A00046 250 ml. pp şişe

A00047 1000 ml. pp şişe

SODIUM HYPOCHLORITE ROOT CANAL IRRIGATION SOLUTION

- Sodium hypochlorite is a 5.25% aqueous solution containing active chlorine.
- It dissolves both living and necrotic tissues in the root canal system.
- It is effective against flora and endotoxins that contaminate the root canal system.
- It efficiently removes debris generated during the chemomechanical instrumentation phase of root canal preparation.
- It eliminates the organic content of the smear layer during the final washing stage of treatment.
- Sodium hypochlorite can be used with EDTA liquid and, after thorough intermediate washes, it can also be used with chlorhexidine.

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A00094 1000 ml. pp şişe

EDTA LIQUID %17 ROOT CANAL PREPARATION LIQUID

- E.D.T.A. effectively binds with calcium and debris, facilitating the removal of demineralized inorganic material and remnants from the canal.
- It helps to quickly drain the canal.
- Its liquid form provides ease of use, thoroughly cleaning the canal walls.
- E.D.T.A. forms chelate-type bonds with calcium apatite in dental tissues, as well as with magnesium and similar structures, making them soluble.
- It enables the easy removal of demineralized organic and inorganic structures from the canal.
- This liquid works by chelating calcium and magnesium in dental tissues, facilitating their dissolution and absorption, which enhances the demineralization of dental particles.

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A00195 1000 ml. pp şişe

PROCHEX %2 CHLOROXIDINE DI GLUCONATE IRRIGATION SOLUTION

- ProChex is an aqueous 2% chlorhexidine digluconate solution, effective against the flora and endotoxins that contaminate the root canal system. It provides sustainable, long-term disinfection of the root canal system with a broad-spectrum bactericidal effect against both Gram-positive and Gram-negative bacteria, as well as an antifungal effect in root canal infections.
- It inhibits MMP activity in the hybrid layer with its chlorhexidine content, protecting the collagen network. This helps increase the adhesion of root canal sealers to the root canal walls.
- ProChex does not contain any surfactant agents, so it does not adversely affect the bond strength of restorations. Additionally, it has low toxicity due to the absence of additives.
- It eliminates microorganisms on the dentine surface in vital teeth.

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A00080 3 gr şırınga x 3 adet

EDTA GEL ROOT CANAL PREPARATION GEL

- It provides easy removal of demineralized inorganic material, debris, and residues from the canal.
- It helps to quickly drain the canal and offers ease of use with its gel form.
- It forms chelate-type bonds with calcium, apatite, magnesium, and similar structures in dental tissues, making them soluble.
- It enhances the penetration of adhesive agents and cements into dental canaliculi.
- It thoroughly cleans dental surfaces.
- It facilitates canal instrumentation in narrowed canals during endodontic treatment.

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A00077 9 gr şırınga x 1 adet
A00078 9 gr şırınga x 2 adet

EDTA GEL (WHITE CREAM) ROOT CANAL PREPARATION GEL

- It provides easy removal of demineralized inorganic material and debris from the canal.
- It helps to quickly drain the canal.
- Its cream form offers ease of use.
- It forms chelate-type bonds with calcium, apatite, magnesium, and similar structures in dental tissues, making them soluble.
- It enhances the penetration of adhesive agents and cements into dental canaliculi.
- It thoroughly cleans dental surfaces.
- It facilitates canal instrumentation in canals affected by attrition during endodontic treatment.

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PROETCH ETCHING GEL

- It is a 37% phosphoric acid solution utilized in enamel and dentin preparation.
- Employed in the preparation of enamel and dentin surfaces following the total etch technique.
- Its blue coloration ensures easy visibility on the surface.
- The gel's appropriate consistency facilitates easy application.

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PROGSOLV GUTTAPERCHA SOLVENT

- Progsolv is a natural aromatic ingredient employed for dissolving gutta percha.
- It serves as a solvent for the removal of gutta percha post-root canal treatment.
- A small drop of Progsolv on a cotton pellet is applied to the pulp chamber where gutta percha needs removal, and after 5-10 seconds, the cotton pellet is removed.
- This process facilitates easy removal of gutta percha.

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AMBALAJ
RE1675 15 gr toz 10 ml. likit

PROCANAL G ROOT CANAL FILLING MATERIAL

- It is an endodontic cement employed alongside Gütta-percha cones for root canal filling.
- Based on zinc oxide eugenol.
- Exhibits bactericidal properties.
- Its homogeneous mixing and creamy texture facilitate easy placement of Gütta-percha in the canal.
- Gütta-percha, once in the canal, ensures thread adhesion and sealing.
- It remains intact without dissolution post-application.
- It does not discolor the tooth, and it can be easily removed.
- Radiopaque for clear visibility on radiographs. It does not discolor the tooth, it can be easily removed.

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PRONOL EUGENOL

- Eugenol consists solely of eugenol derived from clove oil, devoid of any additional agents.
- When combined with zinc oxide, it transforms into a cement typically employed as a base material and temporary filling.

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AMBALAJ
RC0200 10 gr toz 10 ml. likit

PROCAL RADIOOPAQUE CALCIUM HYDROXIDE

- ProCal is a powdered, calcium hydroxide-based temporary canal filler.
- It boasts a high basic pH (>12) and is radiopaque.
- It can be mixed with either pure water or a binder such as glycerin.
- Its fine grains make it easy to mix.
- Primarily utilized as a temporary root canal filling.
- Following chemomechanical preparation and drying with paper cones, ProCal is applied to the root canal by patting it with a base like water or glycerin. It's advisable to allow 7-30 days in the root canal per the treatment protocol.

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AMBALAJ

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PROCAL R RADIOPAQUE CALCIUM HYDROXIDE PASTE

- ProCal R is a ready-to-use, viscous, injectable paste based on calcium hydroxide.
- It boasts a high basic pH (>12) and is radiopaque.
- Its ready-to-use form facilitates easy application.
- Primarily utilized as a temporary root canal filling.
- After completing chemomechanical preparation and drying with paper cones, ProCal R is applied to the root canal. It's recommended to wait for 7-30 days in the root canal as per the treatment protocol.

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PROGLASS F GLASS IONOMER FILLING CEMENT

- PROGLASS F is a chemically hardening glass ionomer filling cement available in powder and liquid forms.
- It's recommended for Class I and Class II restorations on temporary teeth, non-loading permanent teeth, intermediate restorations, and Class I and Class II restorations using the sandwich laminate technique.
- Additionally, it's suitable for Class V and root surface fillings as a base.
- This cement is radiopaque.
- It releases fluoride.

PACKING

RG1135 15 g powder 10 ml. liquid + spoon + mixing ped

RG1136 35 g powder 20 ml. liquid + spoon + mixing ped

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PROGLASS Li GLASS IONOMER LINING CEMENT

- PROGLASS Li is a chemically hardening glass ionomer cement, consisting of powder and liquid components.
- It serves as a base for Class I and Class II restorations on temporary teeth.
- Additionally, it's employed for intermediate restorations and Class I and Class II restorations using the sandwich laminate technique, and as a base beneath all filling materials like composite, amalgam, and indirect restorations.
- Indicated for covering enlarged fissures and small lesions as well.
- This cement is radiopaque.
- It releases fluoride, enhancing protection against dental decay.

PACKING

RG1150 10 g powder 15 ml. liquid + spoon + mixing ped

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PROGLASS LU GLASS IONOMER LUTING CEMENT

- PROGLASS Lu is a chemically cured glass ionomer luting cement, consisting of powder and liquid components.
- It boasts a film thickness of less than 25 µm.
- PROGLASS Lu is suitable for the permanent bonding of crowns, bridges, inlays, onlays, pins, and posts.
- It offers easy application.
- Exhibits low solubility in the mouth.
- Biocompatible.

PACKING

RG1195 15 g powder 10 ml. liquid + spoon + mixing ped
RG1196 35 g powder 20 ml. liquid + spoon + mixing ped

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RS0640 75 gr. toz 20 ml. likit
RS0641 40 gr. toz 10 ml. likit

PROZOE CEMENT ZINCOXİDE EUGENOL CEMENT

- Prozoe Cement, a zinc oxide-eugenol based temporary filling cement, maintains a pH of 7-8.
- The material is conveniently prepared by hand.
- As a temporary filling material, it offers durability against chewing forces.
- Despite its durability, it remains easy to remove.
- It exhibits excellent adhesion to cavity walls.
- Zinc oxide-eugenol based cement is commonly indicated for temporary fillings in dentistry.

PACKING

RS0640 75 g powder 20 ml. liquid
RS0641 40 g powder 10 ml. liquid

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PROPOLY ZINC POLYCARBOXYLAT CEMENT

- PROPOLY is biocompatible and does not irritate the pulp.
- It is a self-liquid activated zinc polycarboxylate cement.
- This cement offers natural adhesion to dentin tissue.
- PROPOLY is suitable for use as a cavity sealant under amalgam and composite fillings.
- To achieve optimal bonding strength, it is recommended to maintain moisture on the tooth surface.

PACKING
RS0520 80 g powder 40 g. liquid + spoon + mixing ped

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PROPHY P POLISHING PASTE

- PROPHY P features a pleasant taste and is saccharin-free.
- Available in various colors and flavors, and does not flow during application.
- Used for detertrage treatment, polishing, and cleaning of teeth.
- Does not cause scratching or abrasion of the enamel during operations.
- Aromas in PROPHY P aid patient tolerance during procedures.
- Effectively removes fine tartar after periodontal cleaning.
- It effectively removes discoloration resulting from consumption of food, beverages, and smoking.
- Indicated for tooth cleaning before composite filling.
- Does not close dental tubules or block connections.
- Minimizes detertrajabrasion as much as possible.

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AMBALAJ
C10625 3 gr şırınga

PRODIAMOND DIAMOND POLISHING PASTE

- PRODiamond is a polishing paste infused with diamond particles suitable for both oral and extraoral applications, ideal for use on both restoration and natural tooth surfaces.
- Enriched with 2-3 micron diamond particles.
- It can be applied to all clinical and laboratory composite materials, resin-containing hard restoration fillings, as well as metal, ceramic, and porcelain applications.

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PROAIR F AIRFLOW POWDER

- PROAIR F - AIR-FLOW Powder is exclusively designed for over-gingival treatments.
- It is utilized for cleaning stains/discoloration and dental plaque, as well as preparing plaque for sealant placement.
- It is effective in removing plaque and stains prior to whitening procedures.
- PROAIR F - AIR-FLOW Powder efficiently cleans plaque and stains in orthodontic patients.
- Additionally, it is employed for cleaning stains to aid in color tone detection.

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AMBALAJ

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PROBAR LC LIGHT-CURING GINGIVAL BARRIER

- PROBAR LC is a light-cured gingival protective barrier.
- It serves as a light-reflecting, passive adhesive (sealing), light-cured resin.
- This methacrylate-based product is easily removable after application.
- It is applied before whitening procedures or in areas where gum protection is required.

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AMBALAJ
A00376 2,5 ml. şırınga + uç

PROPIND PLAQUE INDICATOR GEL

- Plaque indicators like PROPIND ensure the visibility and effective removal of bacterial plaque during tooth cleaning.
- PROPIND is specifically designed for detecting bacterial plaques on the teeth.
- It is distinguished by its pink coloration.

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A00361 2,5 ml. şırınga + uç

PROMARKER C CARIES MARKER GEL

- PROMARKER C is utilized for identifying caries in dentin.
- Caries markers, including PROMARKER C, aid dentists during filling procedures by distinguishing between discolored dentin and carious dentin. While discolored dentin may not be contaminated with bacteria and is softer than healthy dentin, it should not be removed, whereas infected dentin must be thoroughly removed prior to filling.

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PROFILLER UNICOMP UNIVERSAL COMPOSITE

PROFILLER UNICOMP is a polymer-based dental restoration material, a light-cured nano hybrid composite designed for universal use in both anterior and posterior restorations. It is radiopaque and activated by visible light, containing 80% by weight of filler particles ranging from 0.03 to 1.0 µm in size.

Applications of PROFILLER UNICOMP include:

- Direct universal restorations, including occlusal surfaces.
- Repair of porcelain/composite restorations.
- Direct restorations using light-cured composites or compomers.
- Indirect applications such as cavity dressing for pre-treatment purposes.
- Indirect restorations like inlay, onlay, and veneer.
- Treatment of hypersensitivity and exposed root surfaces.

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RK03 0,4 gr x 10 adet kapsül

PROFILLER FLOW FLOW COMPOSITE AESTHETIC

PROFILLER Flow Composite Aesthetic is a radiopaque, flowable composite filling material that cures under visible light for restorative applications. Available in high, low, and medium viscosity forms, it contains 76% filler particles ranging from 0.010 to 0.400 microns in size.

PROFILLER FLOW is suitable for:

- Restoration of tooth structure loss due to caries, trauma, or developmental disorders.
- Class I and II fillings, anterior fillings (class III, IV), class V fillings (cervical caries, root erosions, wedge-shaped defects), and any minor fillings.
- Repair of composite restorations.
- Repair of ceramic coatings and bis-acrylic temporaries.

PACKING

RK02 2 g syringe x 1 pcs + tips (A1, A2,A3,A4, B1,B2, B3,B4,C1,C2,C3,C4,A20, TRANSLUCENT, BW)

RK0202 2 g syringe x 2 pcs + tips

RK03 0,4 g x 10 pcs capsule

PROFILLER GEN8 BOND 8TH GENERATION WITH FILLER

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PROFILLER GEN8 Bond 8th Generation with Filler,

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AMBALAJ

RK0116 5 ml. şişe

PROFILLER GEN8 BOND 8TH GENERATION WITH FILLER

PROFILLER GEN8 Bond 8th Generation With Filler is a self-etching, low-visibility, single-layer, single-bottle universal adhesive designed for use as an adhesive in light-curing restorative treatments. It exhibits excellent adhesion to both dentin and enamel, providing exceptional integrity to both cut and uncut surfaces. With a pH level of around 2.5 when ready for use, it ensures reliable bonding of light or dual-cured composites to various surfaces.

PROFILLER GEN8 is suitable for:

- Bonding light or dual-cured composites to enamel/dentin.
- Application on both cut and uncut enamel.
- Use on both cut and uncut dentin.
- Repair of fractured porcelain/composite materials.

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RK0402 2 gr şırınga x 2 adet + uçlar

PROFILLER FS FISSURE SEALANT

PROFILLER FS Fissure Sealant is a polymer-based dental restoration material specifically designed to safeguard fissures against decay by establishing a leak-proof layer. Its primary function is to prevent food particles or bacteria from infiltrating the fissures, thereby playing a crucial role in caries prevention. This product appears as a fluid, low-viscosity paste available in various colors, which hardens upon solidification.

PROFILLER FS is utilized for:

- Sealing/filling pits and fissures.
- Filling small bruises.
- Sealing milk teeth.
- Addressing margin leakage.
- Modifying temporary fillings and repairing composite restorations.

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RK1002 2 gr şırınga x 2 adet + uçlar

PROFILLER FLOW SELFETCH FLOW COMPOSITE SELF ETCH

PROFILLER Flow Composite Self Etch is a self-etching, self-adhesive, ready-to-use, flowable radiopaque composite material designed for use in light-curing restorative treatments, eliminating the need for preliminary preparation steps.

PROFILLER FLOW SELFETCH is suitable for:

- Repairing composite restorations.
- Blocking and undercut in cavities.
- Fissure closure in molars and premolars with significant deep cracks and high caries risk.
- Addressing tooth material loss due to caries, trauma, wear, or developmental disorders.
- Modifying temporary fillings.
- Extended fissure sealant applications.
- Bottom filling for Class I and Class II restorations.
- Class V restoration.
- Class I and Class III restorations.

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RK06 0,4 gr x 10 adet kapsül

PROFILLER BULK FLOW FLOW COMPOSITE BULK FILL

PROFILLER Bulk Flow Composite Bulk Fill is a low-viscosity, visible light-activated, radiopaque, flowable material. This semi-translucent composite allows polymerization to a depth of 4 mm.

PROFILLER Bulk Flow is utilized for:

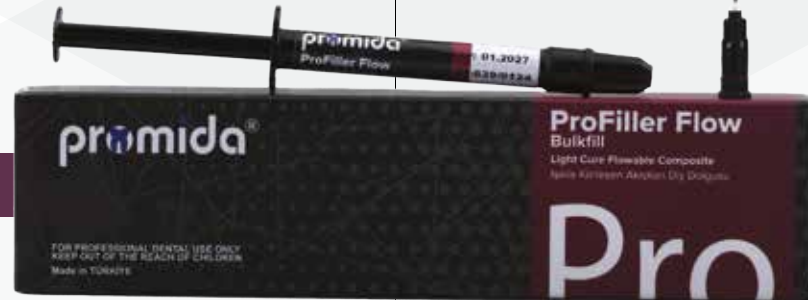
- Pit and fissure sealant applications.
- Minimally invasive cavity restorations, including small, stress-free occlusal restorations.
- Class III and V restorations, such as tooth neck caries, root erosions, and wedge-shaped defects.
- Filling out slips.
- Correcting minor tooth enamel defects.
- Repairing minor defects in aesthetic indirect restorations.
- Repairing resin and acrylic-based temporary materials.
- Core construction in cases where at least half of the crown tooth structure is present to provide structural support for crown placement.
- Restoration of milk teeth.
- Posterior region restorations, including class I and II restorations and single cap renewals.
- Reconstructive applications, such as covering widened fissures in molars and premolars.

PACKING

RK05 2 g syringe x 1 pcs + tips (A1, A2, B1)

RK0502 2 g syringe x 2 pcs + tips

RK06 0,4 g x 10 pcs capsule

PROFILLER DC DUAL CURE CORE BUILD UP

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PROFILLER DC Dual Cure Core Build Up,

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PROFILLER DC DUAL CURE CORE BUILD UP

PROFILLER DC Dual Cure Core Build Up is a dual-curing, radiopaque material specially formulated for dental treatments. It boasts 50% filler particles (ranging from 0.100 to 0.900 microns, Dental Glass) by weight. Its matrix comprises UDMA, Bis-GMA, and other co-acrylates. Upon mixing, it initiates curing (gelling) within 90-150 seconds, with full curing achieved in 3-5 minutes. Utilizing appropriate blue light enables faster curing, with 2 mm cured in 20 seconds and 4 mm in 40 seconds.

PROFILLER DC Dual Cure Core Build Up is suitable for:

- Addressing extensive dental defects, including tooth decay, severe wear, and traumatic injuries.
- Correcting shape and position anomalies of teeth.
- Restoration of tooth loss.

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AMBALAJ

RK09 70 gr çiftli kartuş (A1, A2, B1)

PROFILLER TCB TEMPORARY CROWN BRIDGE

PROFILLER TCB Temporary Crown Bridge is a radiopaque material specifically designed for use in chemically cured dental treatments. Following mixing, it should be removed from the teeth within 90-150 seconds, with full curing achieved in 3-5 minutes. PROFILLER TCB ensures an exceptionally precise fit of the finished temporary restoration with optimal edge integrity. Thanks to its outstanding mechanical properties, PROFILLER TCB is highly resistant to breakage, making it ideal for the fabrication of long-term temporary restorations.

PROFILLER TCB is suitable for:

- Addressing extensive dental defects such as tooth decay, severe wear, and traumatic injuries.
- Correcting shape and position anomalies of teeth.
- Restoration of tooth loss.
- Treating common abrasions.

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AMBALAJ
CM01 50 adet kutu

PROMASK EAR-LOOP SURGICAL MASK

- FILTER (Meltblown)
- Made without glass fiber, hypoallergenic, and consisting of breathable non-woven fabric.
- Nose bar adaptable for a comfortable fit.
- High filtration.
- Latex free.
- Disposable 3-layer surgical face mask with filter (Type IIR).
- Produced and tested according to EN 14683-AC standards.
- Non-sterile.

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AMBALAJ
C10910 80 adet disk + uç

PRODİSK COMPOSITE POLISHING DISK

- Utilized during the finishing and polishing phases of composite surface treatment.
- Features flexible polishing discs coated with aluminum oxide.
- Available in 4 distinct grain sizes, ranging from thick to super thin, and in 2 different disc diameters.
- Each grain size is designated by a specific color for easy identification.
- Equipped with a smart mandrel system for effortless attachment and detachment.
- The shatter-resistant and sterilizable metal mandrel is compatible with discs from any brand.
- Available in customizable quantities and can also be sold as spare discs separately.

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C10915 50 adet 5 mm

PROMATRIX OTO AUTO MATRIX BAND

- A circumferential matrix system designed for posterior teeth.
- Available in 2 different tape thicknesses and 2 different lengths.
- Sold in boxes containing 50 pieces each.
- Dimensions: 0.038 mm thickness with heights of 6.3 mm and 5 mm respectively.

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AMBLAJ
C10918 50 adet matrix + halka

PROMATRIX B SECTIONAL CONTOURED MATRIX BAND

- The safest and optimally contoured locational matrix system, offering the shortest application time.
- It is in boxes of 50 pieces.
- Sold with one ring.

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AMBALAJ
C10916 12 adet

PROMATRIX I IVORY MATRIX BAND

- Ivory matrix bands are crafted from 304 stainless steel.
- Each band is 40 microns thick (0.04 mm).
- Sold in packs of 12.
- 12-13-14-15-16 form numbers are available.

PACKING
C10916 12 pcs

promida®

PROMIDA DIŐ MALZEMELERİ SAN. TİC. LTD. ŐTİ.
Adres: 75. Yıl Sultandere Mah. 3. Cad. (Küçük Org. San.) No:6/A
Odunpazarı / ESKİŐEHİR - TÜRKİYE

www.promida.com.tr

info@promida.com.tr export@promida.com.tr

Tel: +90 222 205 34 84 Export Mobil: +90 542 226 08 81