

INSTRUCTION FOR USE

INSTRUCTION MANUAL

Catalog No: CM01

Components: Breathable non-woven fabric.

Packaging: 50 pieces.

PROMASK – EAR-LOOP SURGICAL FACE MASK

1- Description

Disposable 3-ply filter surgical face mask. Does not contain glass fiber. Hypoallergenic. Made of air breathable non-woven fabric. It has an adjustable nose bar. Latex free.

2- Usage area / Indications

It is used to prevent body fluids passing through the mouth and nose from spreading into the environment.

3- Usage

The folded part of the mask should be on the outside. The ropes on the edges of the mask are passed over the ears and the metal strip is worn over the bridge of the nose.

4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5- Side effects

It has no known side effects.

6- Warning:

It is contraindicated for those who are allergic to the ingredients in its composition. It is for single use only and may cause performance deterioration and cross-contamination if reused.

7- Storage conditions

Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold.

The period of use is 2 years from the date of manufacture.

**** Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.



PROMIDA DIŐ MALZEMELERİ SAN. TİC. LTD. ŐTİ.

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŐEHİR

info@promida.com.tr www.promida.com.tr