

## INSTRUCTION MANUAL

**Catalog No:** RS0505

**Components:** Eugenol

**Packaging:** 10 ml or 20 ml.

## PRONOL EUGENOL

### 1- Description

PRONOL EUGENOL contains only eugenol derived from clove oil, with no additional agents. It is a liquid that, when mixed with Zinc Oxide, forms a cement typically used as a base material and temporary filling.

### 2- Usagearea / Indications

PRONOL EUGENOL is used in making pastes of zinc oxide-based products.

### 3- Usage

Drop a maximum of 2 or 3 drops of PRONOL EUGENOL on a mixing plate, add Zinc Oxide, and mix until the desired consistency is obtained.

### 4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

### 5- Side effects

PRONOL EUGENOL is irritating and may have an allergenic effect due to the eugenol it contains. It may cause sedation. Protect the mucosa with a rubber dam during application.

### 6- Warning:

In case of contact with eyes, skin, or mucous membranes, rinse with plenty of water. Use a rubber dam to prevent it from touching the mucosa. Do not use the solution if there is a known allergy to any of its ingredients. It can only be used by professional dentists. It should not be used with polymerized composite materials. Highly contraindicated for pregnant women when contractions in the uterus are provoked.

### 7- Storage conditions:












Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold. The period of use is 2 years from the date of manufacture.

## NOTE:

Our products are produced for use in dentistry. It is not suitable for use other than by professional dentists. Since the use of the products is under the control of the user other than us, the user is completely responsible for the applications. We recommend using it in accordance with the information and warnings given in the instructions.

**\*\* Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

## 8- Symbols and Meanings:

	Lot number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on Time		Keep away from sunlight
	Warning: Consult the Accompanying Documents.		Refer to the Instructions for Use.
	Storage Temperature Range		Notified body
	Store in a dry place.		