

INSTRUCTION MANUAL

Catalog No: A00361

Components: Caries determining agent, water, glycerin, and excipients

Packaging: 2.5 mL syringe

PROPIND PLAQUE INDICATOR GEL

1- Description

PROPIND is a plaque indicator that ensures bacterial plaques are visible and can be effectively removed during tooth cleaning.

2- Usage Area / Indications

PROPIND is used for the detection of bacterial plaques on teeth.

3- Usage

Isolate the tooth completely. After washing and drying the teeth, apply some PROPIND to the relevant area. Accessories such as brushes can be used to apply the gel to the tooth. Wait 10 seconds and then wash with water. Remove areas that are painted pink.

4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5- Side effects

No known side effects.

6- Warning:

To avoid cross-contamination or infection, do not apply the product directly from the bottle or syringe; use a disposable application tip. Avoid direct contact with skin, eyes, or clothing. In case of contact with eyes, rinse thoroughly with water and consult your doctor.










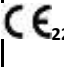

7- Storage conditions: Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold. The period of use is 2 years from the date of manufacture.

NOTE:

Our products are produced for use in dentistry. It is not suitable for use other than by professional dentists. Since the use of the products is under the control of the user other than us, the user is completely responsible for the applications. We recommend using it in accordance with the information and warnings given in the instructions.

**** Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

8- Symbols and Meanings:

	Lot number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on Time		Keep Away from Sunlight
	Warning: Consult the Accompanying Documents		Refer to the Instructions for Use
	Storage Temperature Range		Notified Body
	Store in a Dry Place		