

INSTRUCTION MANUAL

Catalog No: C10910

Components: Aluminum oxide coated material

Packaging: 4 different abrasive grain sizes and 2 different disc diameters. 80 pcs.

PRODISK – COMPOSITE POLISHING DISC

1- Description

PRODISK is used for finishing and polishing composite surfaces. These flexible polishing discs are coated with aluminum oxide and come in 4 abrasive grain sizes (from coarse to superfine) and 2 disc diameters. Each grain size has a distinct color. The smart mandrel system allows for easy attachment and removal of the discs. The break-resistant, sterilizable metal mandrel is compatible with all brands of discs and can be boxed in any desired quantity. Refills are also available for purchase.

2- Usage Area / Indications

PRODISK is used to sand the approximal surfaces of composite, compomer, and glass ionomer fillings. The center sand-free area facilitates entry into interproximal areas.

3- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

info@promida.com.tr www.promida.com.tr

4- Warning:

Avoid swallowing the discs. Ensure that the disc does not become dislodged during use.

5- Storage conditions

Store at room temperature. Protect from moisture, extreme heat, and cold.

NOTE:

Our products are intended for use in dentistry and are not suitable for non-professional use. As the use of these products is beyond our control, the user is fully responsible for their application. We recommend using them according to the information and warnings provided in the user manual.

**** Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.