

INSTRUCTION MANUAL

Catalog No: A00226

Components: Methacrylate Derivative Compounds and Auxiliary Materials

Packaging: 1.5 g Syringe

PROBAR LC – LIGHT-CURING GINGIVAL BARRIER

1. Description

PROBAR LC is a light-cured gingival protective barrier.

2. Usage Area / Indications

PROBAR LC is a light-reflecting, passive adhesive (sealing), light-cured resin. This methacrylate-based resin is easily removed after application.

3. Usage

Remove the syringe cap and securely attach a disposable tip to the PROBAR LC syringe. Insert self-supporting plastic cheek retractors. Wash and dry the teeth thoroughly, paying particular attention to the gingival third. After applying the resin barrier, cure the resin for 20 seconds with a quality curing light using a scanning motion.

Cleaning Process:

After the whitening gel has been fully absorbed, wash the teeth with an air/water syringe and perform suction. Lift the PROBAR LC barrier

from the surface using an explorer or another tool. Remove PROBAR LC from interproximal areas using an explorer and/or dental floss. Wash interproximally with a strong air/water spray.

4. Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5. Side Effects

There are no known side effects.

6. Warning:

Review the instructions and precautions before starting treatment. Keep away from children. Do not use PROBAR LC in patients with known allergies to resins, especially methacrylates. If dermatitis, rash, or any other allergic reaction develops, wash the area with copious amounts of water and consult a doctor. The material should not be injected. Protect the product from heat and sunlight. Before intraoral use, check the flow of material on an inert surface to ensure uniformity. DO NOT continue if resistance is felt; insert a new tip and try again before using it on a patient. Gingiva and oral health should be checked before using this product. If bubbles reappear along the edge of the PROBAR LC, wash the gel and check for leaks. Gently dry the area and reapply PROBAR LC. If the problem persists, remove the drape and reapply.

7- Storage conditions:

Store at room temperature between 5°C – 30°C. Protect from moisture, extreme heat, and cold.












The product has a shelf life of 2 years from the date of manufacture.

NOTE:

Our products are intended for use in dentistry and are not suitable for non-professional use. Since the application of these products is beyond our control, the user is fully responsible for their use. We recommend using the product in accordance with the information and warnings provided in the instructions..

**** Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

8- Symbols and Meanings:

	Lot Number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on time		Keep Away from Sunlight
	Warning: Consult the Accompanying Documents		Refer to the Instructions for Use
	Storage Temperature Range		Notified Body
	Store in a Dry Place		