

INSTRUCTION MANUAL

Catalog No: C10550

Components: Sodium Bicarbonate, Flavor, and Stabilizer

Packaging: 300 g Powder

PROAIR F – AIRFLOW POWDER

1. Description

PROAIR F – AIRFLOW Powder is intended solely for use in over-gingival treatments.

2. Usage Area / Indications

- For cleaning stains, discoloration, and dental plaque.
- For cleaning plaques before the placement of sealants.
- For removing plaque and stains prior to whitening procedures.
- Effective in cleaning plaque and stains in orthodontic patients.
- Used for cleaning stains in color-tone detection.

3. Usage

Before using this product, carefully read the instructions in the manual provided with your device and system. Follow all safety precautions.

 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

info@promida.com.tr www.promida.com.tr

Rev. Tr: 05.01.2023

4. Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5. Side effects

Patients and users with chronic bronchitis or asthma should not be treated with an air polishing device, as sprayed air and dust can cause breathing difficulties. Avoid use in patients with upper respiratory tract disease. Patients on a low-salt diet (e.g., arterial hypertension) should not be treated with this powder due to its sodium bicarbonate content. The flavors in the powder may cause allergic reactions in some cases. Do not direct the spray powder at fillings, dental crowns, or bridges, as this may damage the repair. Avoid use for subgingival treatments. Both patients and users should wear eye protection. Users must wear a mask. For maximum protection, use long cotton rolls inside the mouth. Utilize your dental unit's unloader at high speed.

6. Warning:

Do not exceed the powder filling limit indicated in the dust container of the device you are using. The powder does not contain granulated sugar. Carefully read the instructions in the manual provided with your device and system and adhere to all safety precautions.

7. Storage conditions

Store in a dark place at room temperature between 5°C – 30°C. Protect from moisture, extreme heat, and cold. Close the lid after use to maintain activation. Avoid storing near heat sources or acids. If the bottle is left open, the powder may clump and clog the device.












The product has a shelf life of 2 years from the date of manufacture.

NOTE:

Our products are intended for professional use in dentistry and are not suitable for non-professional use. Since the application of these products is beyond our control, the user is fully responsible for their use. We recommend using the product in accordance with the information and warnings provided in the instructions.

**** Disposal:** Products should be treated as biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

8- Symbol and Meanings:

| | | | |
|---|---|---|-----------------------------------|
|  | Lot number |  | Manufacturing Date |
|  | Catalog Number |  | Manufacturer |
|  | Use on Time |  | Keep Away from Sunlight |
|  | Warning: Consult the Accompanying Documents |  | Refer to the Instructions for Use |
|  | Storage Temperature Range |  | Notified Body |
|  | Store in a Dry Place | | |