

INSTRUCTION MANUAL

Catalog No: C10622

Components: Pumice, flavor, colorant, glycerin, water, and binder

Packaging: Available in packages of 200, 100, and 50 pieces.

PROPHY P – POLISHING PASTE

1- Description

PROPHY P has a pleasant taste and does not contain saccharin in its composition. It has sufficient density in different colors and flavors and does not flow during application. PROPHY P is used for detartrate treatment, polishing, and cleaning of teeth. During operations, it does not cause scratching or abrasion of the enamel. With the help of aromas in its composition, PROPHY P helps the patient tolerate the procedure easily and leaves a pleasant taste in the mouth.

2- Usage Area / Indications

PROPHY P provides the removal of fine tartar after periodontal cleaning. It cleans discoloration caused by food, drink, and cigarette use, helps to lighten teeth color, and is indicated for tooth cleaning before composite filling. It never closes the dental tubules and does not block connections. It can also be used for polishing composite-based fillings, and it helps to easily separate tartar from teeth during

the cleaning process while preventing detartrate abrasion as much as possible.

3- Usage

Take a small amount of PROPHY P on the polishing brush and apply it properly to the tooth surface.

4- Contraindications

Do not use in patients with hypersensitivity to its ingredients.

5- Side effects

No known side effects.

6- Warning:

It is not suitable for use by those who are allergic to the chemicals in its composition. Mouth rinsing can be done after or during treatment. After use, the lid of the paste should be closed, and the product should be protected.












7- Storage conditions

Store at room temperature between 5°C – 30°C as recommended. Protect from moisture and extreme heat and cold. The period of use is 2 years from the date of manufacture.

NOTE:

Our products are produced for use in dentistry. It is not suitable for use other than by professional dentists. Since the use of the products is under the control of the user other than us, the user is completely responsible for the applications. We recommend using it in accordance with the information and warnings given in the instructions.

**** Disposal:** Products should be treated according to their biological hazards and disposed of in accordance with the country's legal regulations and hospital policies.

	Lot number		Manufacturing Date
	Catalog Number		Manufacturer
	Use on time		Keep away from sunlight
	Warning, Consult the accompanying documents.		Refer to the Instructions for Use.
	Storage temperature range		Notified body
	Store in a dry place.		

8- Symbols and Meanings:

 **PROMİDADIŞMALZEMELERİSAN.TİC.LTD.ŞTİ.**

75. YIL MAH. KÜÇÜK ORGANİZE SANAYİ 3. CADDE 6A ODUNPAZARI/ESKİŞEHİR

info@promida.com.tr www.promida.com.tr

Rev. Tr: 01.11.2022